a *Johnson & Johnson* company

THREADED TIBIAL CANNULA

DePuy Mitek
325 Paramount Drive
Raynham, MA 02767, USA
1-800-356-4835

C E0086

EC	REP
----	-----

DePuy International Ltd.
St Anthony's Road, Beeston
Leeds LS11 8DT
United Kingdom
Tel: +44 113 270 0461
Fax: +44 113 272 4101

P/N: 106045 Rev: D 06/05

ENGLISH

MITEK THREADED TIBIAL CANNULA

Rx Only (USA only)

DESCRIPTION

The Mitek Threaded Tibial Cannula is designed for arthroscopic surgical use primarily to plug the tibial tunnel and/or portals. The cannula is tapered and threaded to maintain its place during use over a range of tunnel diameters. The cannula seal is designed to minimize fluid leakage.

MATERIAL

Tunnel Plug: Polyethylene

Dam/Seal: Silicone

INDICATIONS

Intended for arthroscopic surgical use to plug the tibial tunnel and/or maintain portals during insertion or extraction of instruments.

CONTRAINDICATIONS

Procedures other than arthroscopic techniques are contraindicated.

WARNINGS

Do not resterilize. The Mitek Threaded Tibial Cannula is provided sterile, for single use only. Discard any open, unused product.

PRECAUTIONS

1. A surgeon should not begin clinical use of the Mitek Threaded Tibial Cannula without reviewing the instructions for use.
2. Wear eye protection during the use of this device.

INSTRUCTIONS FOR USE

1. Gently thread the tapered Threaded Tibial Cannula into the previously drilled tunnel until resistance is felt.
2. Instruments may be inserted through the seal without significant loss of fluid.

STERILIZATION

The Mitek Threaded Tibial Cannula is provided sterile. Do not attempt to resterilize.

STORAGE

Store in a cool, dry place (below 80°F or 26°C). Once opened, the device should be used in surgery or discarded. Do not use after expiration date.

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All rights reserved.